English
Español

Owner's Manual Manual del Propietario

Bio Free Ultrasonic Cleaner 6L
Lavadora Ultrasonica Bio Free 6L

Cod. 300054261Rev.00

GNATUSO

INDEX

PRESENTATION OF MANUAL	04
IDENTIFICATION OF EQUIPMENT	05
- Identification of the Equipment	05
- Principles and bases applied to the operation of the product.....	06
- Description of the Equipment	06
- Indication of Equipment	06
MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION	07
APPLICATIONS	08
- Application examples	08
- Items not suitable for ultrasonic cleaner	08
TECHNICAL SPECIFICATIONS	09
- Technical specifications of the product and its accessories	09
- Parts lists and circuit diagrams	11
- Symbolologies of packaging	11
- Symbolologies of product	11
- Contents of the accessible and inaccessible markings	12
OPERATION OF EQUIPMENT	13
PRECAUTIONS, RESTRICTIONS AND WARNINGS	21
- Conditions of transport and storage	22
- Sensitiveness to environmental conditions foreseeable in normal situations of use	22
- Precautions and warnings "during the installation" of equipment.....	22
- Recommendations for preserving the equipment.....	22
- Precautions and warnings "during the utilization" of equipment	23
- Precautions and warnings "after" the use of equipment	23
- Precautions and warnings during the "cleaning and disinfection" of equipment.....	23
- Precautions in case of alteration in the functioning of equipment.....	24
- Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment.....	24
CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION.....	24
- Additional procedures for reuse	24
- Cleaning and disinfection.....	24
- Preventive maintenance	25
- Corrective maintenance.....	25
UNFORESEEN EVENTS – SOLUTION OF PROBLEMS	26
WARRANTY OF EQUIPMENT	27
FINAL CONSIDERATIONS.....	27

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PRESENTATION OF MANUAL

INSTRUCTIONS OF USE

Technical name: Ultrasonic Cleaner

Brand: Gnatus

Trade name: Bio Free Ultrasonic Cleaner 6L

Manufacturer/ Distributor:

GNATUS - EQUIPAMENTOS MÉDICO-ODONTOLÓGICOS LTDA.
Rod. Abrão Assed , Km 53+450m - Cx. Postal 782 CEP 14097-500
Ribeirão Preto - S.P. - Brasil
Fone +55 (16) 2102-5000 - Fax +55 (16) 2102-5001
C.N.P.J. 48.015.119/0001-64 - Insc. Est. 582.329.957.115
www.gnatus.com.br - gnatus@gnatus.com.br
Technical Duties: Gilberto Henrique Canesin Nomelini
CREA-SP: 0600891412

"Declared exempt from registration by the Ministry of Health"

ATTENTION

For greater safety:

Read and understand all the instructions contained in these Instructions for Use before installing or operating this Equipment.

Note: These Instructions for Use must be read by all the operators of this Equipment.

IDENTIFICATION OF THE EQUIPMENT

Dear Customer

Congratulations. You have made a good choice when you decided to buy a GNATUS QUALITY product comparable to the best products available in the World. This manual is a general presentation of your product and it will give you important details to help you to solve possible problems.

Please, read it and keep this with you.

Identification

Technical name: Ultrasonic Cleaner

Trade Name: Bio Free Ultrasonic Cleaner 6L

Brand: GNATUS

IDENTIFICATION OF THE EQUIPMENT

Principles and bases applied to the operation of the product

Cleaning occurs by cavitation, reaching uniformly throughout the material to be clean, even in hard to reach places.

This occurs through an ultrasonic vibration generator, where a field test is subjected to a vibration in high frequency when immersed in liquid solution.

The vibration generates a negative pressure causing the emergence of micro bubbles in the liquid and a positive pressure providing the collapse of these bubbles on the surface of the proof body forming micro jets that are responsible for erosion and cleaning effect.

Description of the Equipment

Ultrasound equipment for cleaning medical and dental instruments and accessories composed of:

- Injected body in ABS;
- Control panel with digital display;
- Transparent and removable viewer cover,
- Tank made of stainless steel bowl and plastic basket;
- Ultrasonic vibration generators;
- Power cord.

Indication of Equipment

Medical, dental and laboratory instruments and supplies cleaning "removal of blood and small particles on the instruments prior to disinfection". It should be used and handled by qualified person (professional properly regulated, according to local legislation of the country) following the instructions contained herein.

It is the user's obligation to use only the equipment in perfect conditions and protect himself/herself, patients and others from possible dangers.

This equipment performs the cleaning of instruments and does not perform disinfection and/or sterilization processes.

MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION


The content of this page is for information purposes, and the equipment can be shown different. So, when purchasing, check the technical compatibility between the equipment, couplings and accessories.

The Drawing illustrates the equipment with all the optional items. Your delivery unit will only be composed of the items chosen during your purchase option.

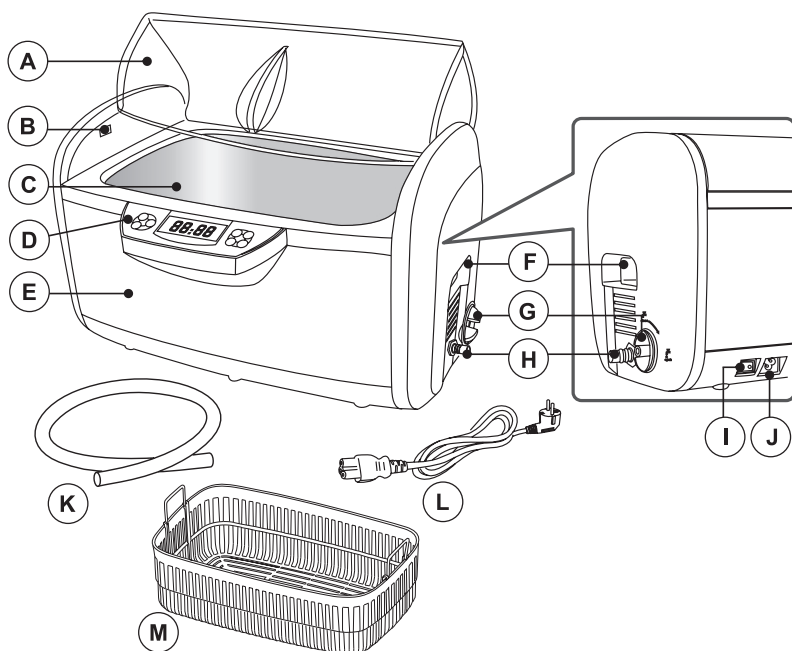The use of any part, accessory or materials not specified or referred to in these instructions is the user's responsibility.


- A - Cover
- B - LED Lights
- C - Stainless steel bowl
- D - Control Panel
- E - Cleaner Body
- F - Handles
- G - Drain Valve

- H - Drain tube
- I - Power Switch
- J - Power cord input
- K - Hose
- L - Input power cord
- M - Plastic basket

APPLICATIONS

Application examples

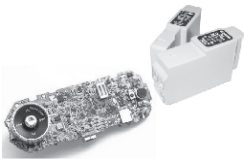Debris left in small holes and cracks are difficult to clean, where bacteria and fungi can develop. Use ultrasonic cleaning to deep clean items prior to disinfection.

Precision parts	Articles that use PCBs	Personal items
		
Bearings, gears, valves, fuel injectors, etc.	PCBs cell phones, MP4, ink cartridges, etc.	Glasses and sunglasses.
Medical / dental instruments	Laboratory articles	
		
Surgical instruments, pliers, drills, hand pieces, etc.	Test tubes, beakers, bottles, etc.	

Items not suitable for ultrasonic cleaner

Mild jewelry, pearls, emeralds, ivory, coral, agate, tortoise shells, etc.	These objects have little strength, and scratches may occur during the cleaning.
Welded items, chromed and pasted: Weld metal, galvanized metal, connections.	The ultrasonic cleaner can increase the spacing of welds, plated or glued items, and can cause separation.

APPLICATIONS

Items not suitable for ultrasonic cleaner

Other: Glassware, ceramics, filters, preexisting cracks.

The ultrasonic cleaner can increase the preexisting cracks in the coating of glasses and ceramics. If the items do not have preexisting cracks, so there is no problem.

TECHNICAL SPECIFICATIONS

Technical specifications of the product and its accessories

Features:

High capacity tank:

Tank capacity 6000 ml

Two industrial ultrasonic transducers:

More even distribution of sound waves, deep cleaning and high durability

Two-color LED display:

Clock with a range from 10 to 30 minutes, five temperature settings, operating status indicators

Two ceramic heaters:

Modern, safe, efficient and durable

Independent control circuits for each ultrasonic transducer:

High efficiency, not susceptible to interference

Multiple circuits protectors:

When overloaded or misused, the guards stop the power supply protecting the equipment

Cooling fan:

Improves heat dissipation and benefits continuous operations

Degassing feature:

Dissolves the solution quickly

Drain Valve: Easy to empty

General

Model:

Bio Free Ultrasonic Cleaner 6L

Equipment Classification according to ANVISA: Class I

Operating Mode: Continuous operation with intermittent load On - 45 min - Off - 20min

TECHNICAL SPECIFICATIONS

Power

Power Supply:
127 or 220 V~
Frequency:
50/60 Hz
Number of stages:
Monophasic / Biphasic
Power:
310W (220V), 290W (127V)

Other specifications

Ultrasonic Frequency:
35.000Hz
Tank capacity:
6000 ml
Tank Dimensions:
32,3 x 20,5 x 9,9 cm
Longer object allowed in the tank:
34,0 cm
Digital clock setting:
Clock with 1-30 minute interval
Tank material:
Stainless Steel
Temperature settings:
40°C - 45°C - 50°C - 55°C - 60°C
Body Material (housing):
ABS
Net weight:
5,4 kg
Gross weight:
6,8 kg
Unit Dimensions:
43,8 x 30,0 x 35,0 cm
Packaging dimensions:
50,5 x 36,0 x 32,4 cm

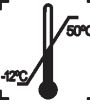
TECHNICAL SPECIFICATIONS

-  Pay attention when using this equipment together with other equipment that may move, in order to avoid collisions.
-  The materials used to produce the equipment are Biocompatible.
-  The parts of this equipment are not subject to degradation by wear, corrosion, fatigue or aging of materials during their lifetime.

Parts lists and circuit diagrams

Gnatus states that the provision of circuit diagrams, parts list or any other information that will provide technical assistance by the user, can be requested, if previously agreed between the Company and Gnatus.

Packing symbols

- | | | | |
|---|---|--|--|
|  | <p>Maximum stacking: It determines the maximum quantity of boxes which can be stacked during transportation and storage "as per packaging".</p> |  | <p>Packing to be transported and / or stored avoiding humidity, rains and wet floor.</p> |
|  | <p>Packing to be transported and / or stored with the harrows up.</p> |  | <p>The packing must be stored and transported away from direct sun light exposure.</p> |
|  | <p>Packing to be transported and / or stored with care (should not suffer drop and neither receive impact).</p> |  | <p>Temperature limit for the packing to be stored or transported.</p> |

Product symbols

- | | | | |
|---|--|---|---|
|  | <p>Careful : It indicates an important instruction for the operation of the product. Not following it can cause dangerous malfunctioning.</p> |  | <p>Indicator Light - Working Status</p> |
|  | <p>Note: It indicates useful information for operation of the product.</p> |  | <p>Function button - Functions (Degassing/Normal)</p> |
|  | <p>Clock quick setup button</p> |  | <p>On/Off button</p> |

Product symbols

Important : It indicates an instruction of safety for operation of the product. Not following it, can lead to serious danger to the patient.

Digital display

Grounding (at several points of the equipment) indicates the condition of being grounded.

Degassing status:

Warning - see the manual

Normal cleaning status

Heater status

Temp

Temperature setting button

Time setting button

TC


Heater power button

Content of accessible and non-accessible demarcations

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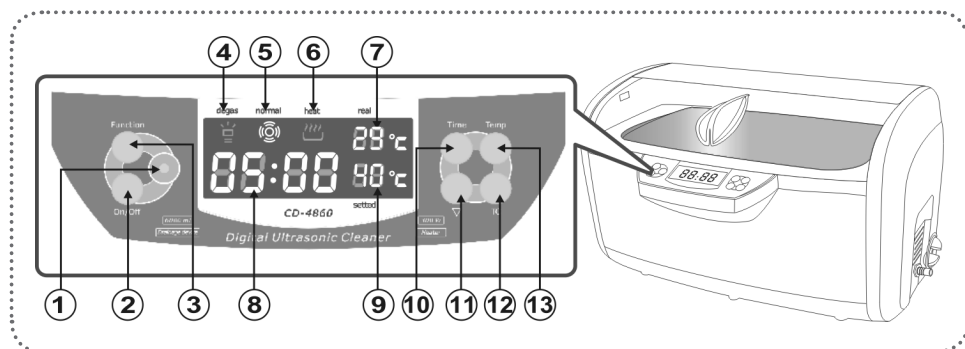English

1. Indicator light – Working status

Green - Normal operating status, ready for use.

Red - Under the protection of overheating, it takes 15 minutes of waiting for the light changes from red to green. The operation can be resumed later.

2. On/Off button – on/off switch on the unit

After turning on, the LED display, with standard time , and the unit will be ready and with normal operating status.

Press the On/Off button once to start the operation of the machine. Cleaning stops when the countdown stops . If the device needs to be stopped before the time runs out, press the On/Off button.

3. Function button - Functions (Degassing/Normal)

Press the Function Button (03) , it lights up, indicating that the degassing function has been selected. Press the On/Off button to start the degassing function . It illuminates for about 2 minutes, then the unit will return to the normal cleaning function (05). Pressing the On/Off button before the end will cause the degassing to be interrupted. If you need additional degassing, press the Function Button again (03), .

Degassing

The newly added water can generate many air bubbles on the walls of the tank. This circumstance reduces the effect of cleaning in the early phase of the ultrasonic cleaning. Turning on the degassing function dissipates the droplets and bubbles of air, improving the efficiency of cleaning (approximate time, 2 minutes).

4. - Degassing status

 - Illuminated, the degassing function is selected.  - Blinking, the degassing is in progress.

EQUIPMENT'S OPERATION

Control panel and operations

5. - Normal cleaning status

 - Illuminated, normal cleaning is in progress.

6. - Heater status

Press the TC button, it flashes  indicating that the heater is on. Press the TC button again, it turns off , indicating that the heater is off.

7. Water temperature display. Displays actual water temperature in the tank.

8. 05:00 Operating time display. It counts down, after the work is started.

9. Water temperature display (configuration). Press the "Temp" button to select one of five temperature settings.

10. TIME button (configuration). Clock quick setup button.

Press the TIME button, the LED displays 05:00. Press the button again to set the clock. Each time pressed, the button adds 5 minutes.

11. Button  - Clock decrease button. Each time pressed, reduces the clock in 1 minute.

12. CT Button - Heater button. Press the TC button, when flashing , it indicates that the heater is on. To cancel the heat, press TC again, , it turns off indicating that the heater is off.

13. TEMP button - Temperature adjustment button. 5 Temperature settings can be selected. The standard temperature is 40°C. Press the "TEMP" button to select the temperature in the following sequence:

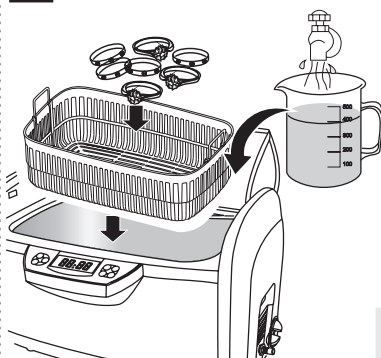
When the water temperature reaches 60°C, the system turns off automatically.

EQUIPMENT'S OPERATION

Cleaning methods

1 Regular cleaning

Use tap water, chemical solutions are unnecessary in most cases.

Only tap water is required.

Appropriate places: Optical stores, laboratories chemical and dental clinics.

Cleaning methods:

1. Add water to the cleaning tank to a level between the lines "MIN" and "MAX" and above the object.
2. Place the objects to be cleaned inside the basket and place the basket inside the tank.
3. Turn the main switch on. Press the TIME button to set the clock between 5 and 10 minutes. Press the On/Off button once to start the cleaning.

Notes on the use of the basket:

1. The basket reduces friction between the objects and the tank, but absorbs about 30% of ultrasonic energy reducing the cleaning effect.
2. A metal basket absorbs less ultrasonic energy (about 8%).

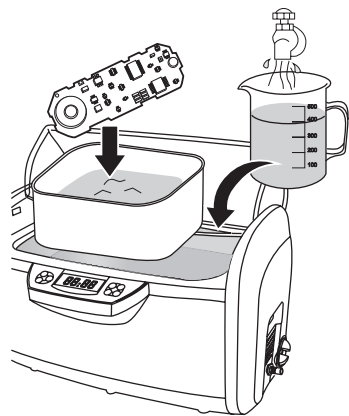
This model is equipped with a large tank. When cleaning small objects, the method of "indirect cleaning" can be used with satisfactory results. Ultrasonic cleaners can also do the job.

EQUIPMENT'S OPERATION

Cleaning methods

2 Indirect clearing

Place objects in a separate container. Use ultrasonic waves to penetrate the container and to clean.

Appropriate places: Print shops, dental clinics, technical assistance of mobile phones and electronics.

Different industries use different fluids to be used inside the containers:

1. Sterilization of medical instruments - Special Solution (cleaning before sterilization).
2. Precision parts - Hexane (to dissolve fat to evaporate quickly).
3. Technical assistance for mobile phones and electronics - pure alcohol (to evaporate quickly).
4. Print shops - Acetone is often used (for dissolving the paint).

Do not allow contact between acetone and the plastic parts of the equipment to avoid corrosion. Use indirect clean and a glass or metal container for acetone.

Cleaning methods:

1. Place the object inside a container. Add the proper fluid to submerge the object.
2. Place the container directly into the tank. Add water to a level between MIN and MAX, without covering the container.
3. Press the TIME button to set the clock to 10 minutes. Press the On/Off button once to start cleaning. The ultrasonic waves will penetrate the container and clean the object.

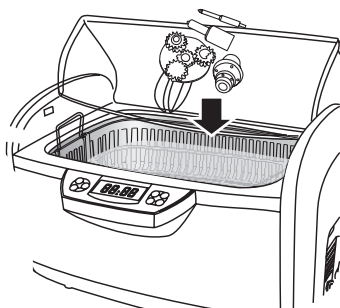
Choose the container:

1. Plastic packaging - plastics are soft, and absorb about 30% of ultrasonic energy reducing the effect of ultrasonic cleaning.
2. Aluminum containers - absorb about 20% of ultrasonic energy.
3. Stainless steel containers - absorb about 8% of ultrasonic energy.

EQUIPMENT'S OPERATION

Cleaning methods

3 Advanced cleaning

Debris accumulated over a long period of time, very dirty or greasy objects.

Appropriate places: Automotive repair and hardware manufacturers stores.

Cleaning methods:

1. Large objects can be placed directly into the tank. Small objects can be placed in the basket and then into the tank.

Do not place objects on each other, to avoid friction during cleaning.

2. Add water to a level between MIN and MAX and above the objects.

3. Turn the main switch on. Press the TEMP button to set the temperature of the water to 45°C.

Press TC to turn on the heater , it flashes. Feche a tampa. Close the lid. The LED display shows the actual temperature of water. When the water reaches the set temperature,  it turns off indicating that the device is off.

4. Press the TIME button to set the clock to 10 minutes. Press the On/Off button once to start cleaning. The fat will be dissipated and appears to be something as a black smoke on the water.

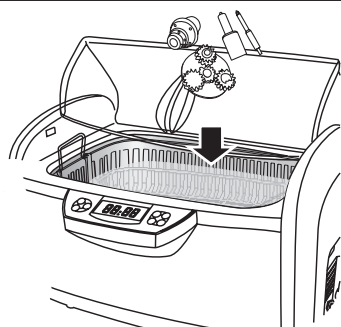
5. When it stops, open the drain valve to release the dirty water. Close the valve later.

6. Clean the tank and replace the water. Wash objects for more 3 minutes to remove residual debris and detergent.

EQUIPMENT'S OPERATION

Cleaning methods

4 Special cleaning

Objects made of silver, copper or brass darkened by oxidation need special cleaning.

Appropriate places: Manufacturers of products made of copper, silver or bronze.

Cleaning methods:

1. Place the objects in the basket and then in the tank; add water to a level between MIN and MAX above the objects.

2. Turn the main switch on. Press the TEMP button to set the temperature of the water to 50°C. Press the TC button to turn on the heater,  it blinks. Close the lid. The LED display shows the actual temperature of water. When the water reaches the set temperature, it turns off  off indicating that the device is off.

The heater has dual protection. It will be disconnected automatically after 50 minutes.

3. Press the TIME button to set 15 minutes for the clock.

4. Press the function button to select the degassing function. The button lights . Press the On/Off button to start the degassing. After 2 minutes the device switches the mode for normal cleaning automatically.

5. When cleaning is complete, remove the basket and objects. If there are many objects and some debris remaining, add 5 minutes and repeat the cleaning.

6. Follow steps 5 and 6 in the advanced cleaning section to drain the dirty water, add clean water, wash objects for more 3 minutes to remove the residual chemicals.

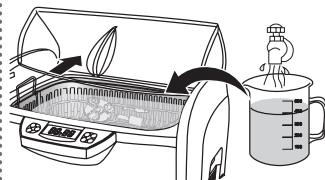
During cleaning the debris separate from the objects as "smoke". The water becomes blurred. This method will remove the rust and will cause objects to appear as new.

EQUIPMENT'S OPERATION

Operating guidelines

1


Open the lid, as shown, place the objects in the basket and then place them in the cleaning tank. Add water to a level between "MIN" and "MAX" and above the objects.

If the device is switched on without water, the ultrasonic energy will not be absorbed. After more than 15 seconds, this procedure can damage the device or drastically reduce its lifetime.

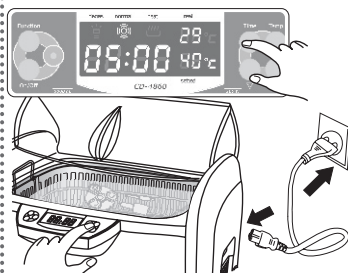2


Connect the power cord into a power outlet and turn the switch on. The LED display shows **05:00**.

This is the most common configuration of the clock and or operating status. If the clock needs to be adjusted, press the TIME button ▽. (As required) The clock can be set between 1 and 30 minutes. The periods of common cleaning are 5, 10 and 15 minutes.

The extension of the cleaning period can result in:

- Loosening the screws (if used).
- Increased preexisting cracks.
- Peeling on the lining of the object.

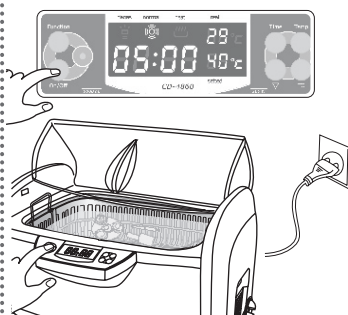3

Press the On/Off button once to start the cleaning.

During cleaning, the two blue LEDs from the unit will light, a humming can be heard coming from the cleaning tank, indicating that cleaning is in progress. Closing the lid will reduce the noise level.

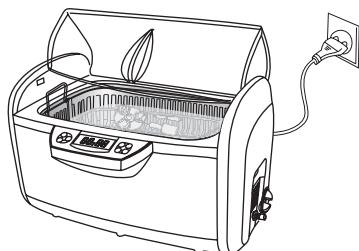
The LED display will count down, showing the remaining cleaning time. When the display reads 0:00, the cleanup is complete. To interrupt the cleaning at any time, press the On/Off button.

The unit is designed with overheating protection, to avoid extended and continuous operation for long time. When the red light is on, the device cannot be activated, even if it is forced to do so. The unit remain idle for about 20 minutes and can be restarted after the green light illuminates.

EQUIPMENT'S OPERATION

Operating guidelines

4

Select one of four cleaning methods previously recommended.

a. When the heater is needed, press the TEMP button to select one of five preset temperature settings.

Press the TC button, it will flash , indicating that the heater is on. The LED display shows the actual temperature of water. When the water reaches the set temperature, it turns off , indicating that the heater is off.

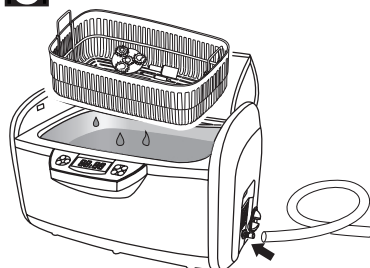
To ensure safety, the unit is designed with double protection.

The heater will automatically shut down if running for 50 minutes.

b. The degassing function can be used. Press the TIME button to set the clock to 10 minutes, and then press the function button, it will light up . Press the On/Off button to start degassing. The device switches to the normal cleaning mode after 2 minutes.

c. When the "smoke" of the waste is no longer visible, the cleanup is complete. If further cleaning is required, set the clock again and repeat the above steps.

5


When cleaning is complete, disconnect the power cord, open the cover and remove the basket and objects.


Connect the hose to the drain pipe, as previously recommended.

- Open the drain valve to drain the dirty water.
- Clean and dry the tank.
- Close the drain valve.

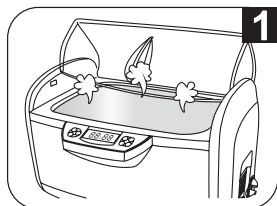PRECAUTIONS, RESTRICTIONS AND WARNINGS

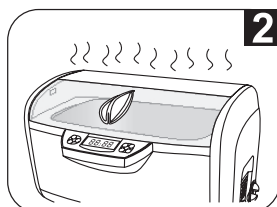
Do not connect the device without water in the tank.
Even if the unit has been designed with multiple protections, if it is turned on for more than 15 seconds without water in the tank, it could damage it or reduce dramatically its lifetime.


Do not leave the unit in operation for long time or continuously.

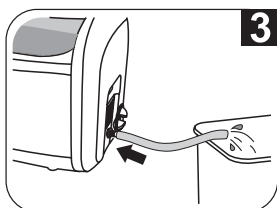
The unit is designed to protect against overheating. If the unit has been running for 45 minutes, it is recommended to stop it for about 20 minutes.

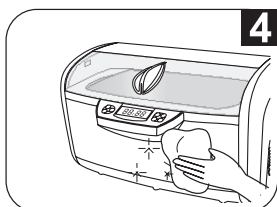Do not keep water in the cleaning tank for long time.
After cleaning is finished, open the drain valve to drain the dirty water. Clean and dry the tank.

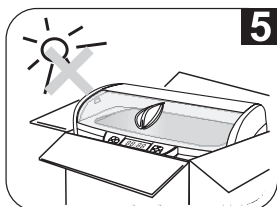Do not spray water on the equipment.
Use a towel to dry the tank and the equipment body.

Do not expose the device to direct sun light for a long period.
Keep the unit in a dry, cool and ventilated place.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Transportation and storage

This equipment must be transported and stored observing the following directions:

- Avoid falls and impacts;
- Keep it dry, do not expose it to rain, water drops or wet floor;
- Keep it away from water and direct sunlight, and in its original wrapping;
- Don't move it over irregular surfaces, protect it from rain and observe the maximum stack quantity specified in the packaging;
- Transportation and storage temperature range: -12°C to 50°C;
- Transportation and storage relative humidity range: 0°C to 90°C;

The Equipment maintains its condition of safety and efficacy, provided that it is maintained (stored) as mentioned in this instruction of use. Thus, the equipment will not lose or alter its physical and dimensional features.

Sensitivity to environmental conditions in normal situations of use

- The equipment has been planned not to be sensitive to interference such as magnetic fields, external electrical factors, electrostatic discharge, pressure or variance of pressure, provided that the equipment is installed, maintained, clean, preserved, transported and operated as per this instruction for use.

Precautions and warnings "during the installation" of equipment

- Make sure that the voltage of the equipment is in accordance with the local network (127 or 220V).
- Position the equipment in a place where it will not get wet.
- Install the equipment in a place where it will not be damaged by pressure, temperature, humidity, direct sunlight, dust, salts, or sulfur compounds.
- This equipment was not planned for use in an environment where vapors, anesthetic mixtures inflammable with air, or oxygen and nitrous oxide can be detected.
- Check the voltage of the equipment at the moment of executing the electrical installation.
- The equipment must be grounded correctly.
- Before the first use and/or after long interruptions from work such as vacations, clean and disinfect the equipment; eliminate air and water deposited in the internal hoses.

Grounding:

Grounding is compulsory, only this way the user will be completely safe. It is only the user's responsibility not to ground, implying in the loss of warranty. Use three-phase outlet. Never connect the grounding to the neutral.

Recommendations for the dental equipment maintenance

Your Gnatus equipment has been designed and developed according to the standards of modern technology. Similarly to other kinds of equipment, it requires special care, which is many times neglected due to several reasons and circumstances.

Therefore, here are some important reminders for your daily routine. Try to follow these simple rules, which will save you a lot of time and will avoid unnecessary expenses once they start making part of your working procedure.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Precautions and warnings “during the use” of equipment

- The equipment should only be operated by duly enabled and trained technicians (Dental Surgeons, Capacitated Professionals)
- If any maintenance should be required, only use services of the Gnatus Authorized Technical Assistance.
- The equipment has been manufactured to handle both continuous and intermittent operation; so follow the cycles described in these Instructions for Use.
- Although this equipment has been planned in accordance with the standards of electromagnetic compatibility, it can, in very extreme conditions, cause interference with other equipment. Do not use this equipment together with other devices very sensitive to interference or with devices which create high electromagnetic disturbance.
- Do not expose the plastic parts to contact with chemical substances, use in the routines of dental treatment, such as: acids, mercury, acrylic liquids, amalgams, etc.
- Never block the ventilation openings of the device. Keep openings free of lint, hair and other materials.
- Do not use the product without filling the tank with water. The dry running can damage the unit.
- Do not fill the tank above the maximum to avoid overflow.

Gnatus shall not be responsible for:

- Use of the equipment differing from that for which it is intended.
- Damages caused to the equipment, the professional and/or the patient by the incorrect installation and erroneous procedures of maintenance, differing from those described in these Instructions for use which come with the equipment or by the incorrect operation of it.

Precautions and warnings “after” the use of equipment

- Turn off the main switch of the dental set when it is not in use for an extended period of time.
- Always maintain the equipment clean for the next operation.
- Do not modify any part of the equipment. Do not disconnect the cable or other connections without need.
- After using the equipment, clean and disinfect all the parts which may be in contact with the patient.

Precautions and warnings during the “cleaning and disinfection” of equipment

- Before cleaning the equipment, turn off the main switch.
- Avoid spilling water, or other liquids inside the equipment, which could cause short circuits.
- Do not use microabrasive material or steel wool when cleaning, or employ organic solvents or detergents which contain solvents such as ether, stain remover, gasoline etc.
- When cleaning of silver or copper, where the effect of oxidation caused a blackout, you must add special solutions to the water to remove rust.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Precautions in case of alteration in the functioning of equipment

- If the equipment has any abnormality, check if the problem is related to any item listed in the topic of unforeseen events (failures, causes and solutions). If it is not possible to resolve the problem, turn off the equipment, remove the power supply cable from the socket and contact your representative (Gnatus).

Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment

In order to avoid environmental contamination or undue use of the Equipment after it has become useless, it should be discarded in the suitable place (as per the local legislation of the country).

- Pay attention to the local legislation of the country for the conditions of installation and disposal of residue.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Additional procedures for reuse

The equipment can be reused in undetermined, i.e. unlimited, quantities, only needing to be cleaned and disinfected.

Cleaning and disinfection

Important: In order to execute cleaning or any type of maintenance, ensure that the equipment is disconnected from the electrical network.

The cleaning procedure below should be made to start work and after each patient.

Always turn off the main switch before performing the daily maintenance procedures

After the exchange of fluid in the stainless steel bowl, remove the waste deposited in the bottom by using a soft cloth or sponge. Dry and clean by using a cloth moistened with alcohol 70°.

To clean the body, including the cover, use a cloth moistened with neutral soap and water or the product recommended below:

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

To clean the equipment, we recommend the use of "BactSpray" (Reg nº MS: 3.2079.0041.001-5) or any other similar product:

Active component:

Benzalkonium chloride (tri-quaternary ammonium)

Solution 50%..... 0.329%

Chemical composition: Butyl Glycol, Decyl polyglucose, Sodium Benzoate, Sodium Nitrate, Essence, Deodorized Propane / Butane, demineralized Water.

For more information concerning cleaning procedures, see manufacturer's instructions.

ATTENTION:

- This product can also be used for cleaning and disinfection of the water unit basin.
- In order to eliminate safety risks or damage to equipment, we recommend avoiding liquids from getting inside the equipment when cleaning.
- The application of other solvent-based or sodium hypochlorite chemical products to clean is not recommended, since they can damage the equipment.

English

NOTE: The registration at the Ministry of Health of the "BactSpray" is executed separately from the product described in this manual, as the "BactSpray" is not manufactured by Gnatus.

NOTE: Use gloves and other systems of protection, during the disinfection.

Preventive Maintenance

In order to protect your equipment, seek Gnatus technical assistance for periodic revisions of preventive maintenance.

Corrective Maintenance

If the equipment has any abnormality, check if the problem is related to any of the items listed in the item Unforeseen Events (situation, cause and solution). If it is not possible to solve the problem, turn off the equipment, and request Gnatus technical assistance.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

UNFORESEEN EVENTS – SOLUTION OF PROBLEMS

 Upon coming across any problem in operation, follow the instructions below to check and repair the problem, and/or get in touch with your representative.

Problem	Probable cause	Solution
-Completely dead.	-Plug disconnected from network.	- Connect the plug.
	-Equipment fuse burned out.	-Disconnect the power supply of the equipment and request the presence of a Licensed Technician.
	-Lack of power in the network.	-Wait the network normalization.
-Little or no cavitation.	-Equipment connected on the inappropriate voltage.	-Ligar a lavadora na voltagem correta da rede. Caso o equipamento for 110V e foi ligado na rede 220V, o mesmo sofreu queima, entrar em contato com Assistência Credenciada.
	-Ultra generator burn. -Incorrect voltage installation or operating without water in stainless steel bowl.	-Contact the Authorized Support.
-Water heating system does not work.	-Burned resistance.	-Contact the Authorized Support.

WARRANTY OF EQUIPMENT

This equipment is covered by the warranty terms counting from the date of installation, as specified below; provided that the defect has occurred in normal conditions of use and that the equipment has not remained stored for more than 06 months counting from the issue date of the sales document until the date of the actual installation.

- WARRANTY TERMS: TWELVE MONTHS;

- LOSS OF THE WARRANTY:

- A) Attempt to repair using an inadequate tool or by unauthorized technicians;

- B) Installation of the equipment by an unauthorized technician;

- C) Damage arising from inappropriate storage or signs of infringement;

- D) Incorrect use of the equipment;

- E) Use of a cleaning product not indicated by the factory;

- F) Falls or blows which the equipment may undergo or lack of observation of an compliance

with the guidelines of the Owner's Manual, which was delivered with the present document, together with the equipment. Repair or replacement of parts during the warranty period shall not extend the validity term of their warranty.

- This warranty does not exempt the customer from paying the service charge for the visit and the travel expenses of the technician, except when the customer sends the equipment to execute the maintenance inside the establishment of the technical assistance.

"Consumer Defense Code - art. 50, unique paragraph".

- The Warranty Certificate comes with the product and must be filled in upon the date

of installation by the Gnatus Authorized Technician.

- Queries and information: GNATUS Help Desk (+55) 16 2102-5000.

FINAL CONSIDERATIONS

The most important aspect related to equipment care is that concerning spare parts.

To guarantee the life span of your equipment, use only original Gnatus spare parts. They are sure to follow the technical specifications and standards required by Gnatus.

We must also point out to you our chain of authorized dealers. Only dealers that make part of this chain will be able to keep your equipment constantly new for they count on technical assistants who have been trained and on specific tools for the correct maintenance of your equipment.

Doubts and information: GNATUS Call center (55-16) 2102-5000.

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